


Genius

Genius Proflex

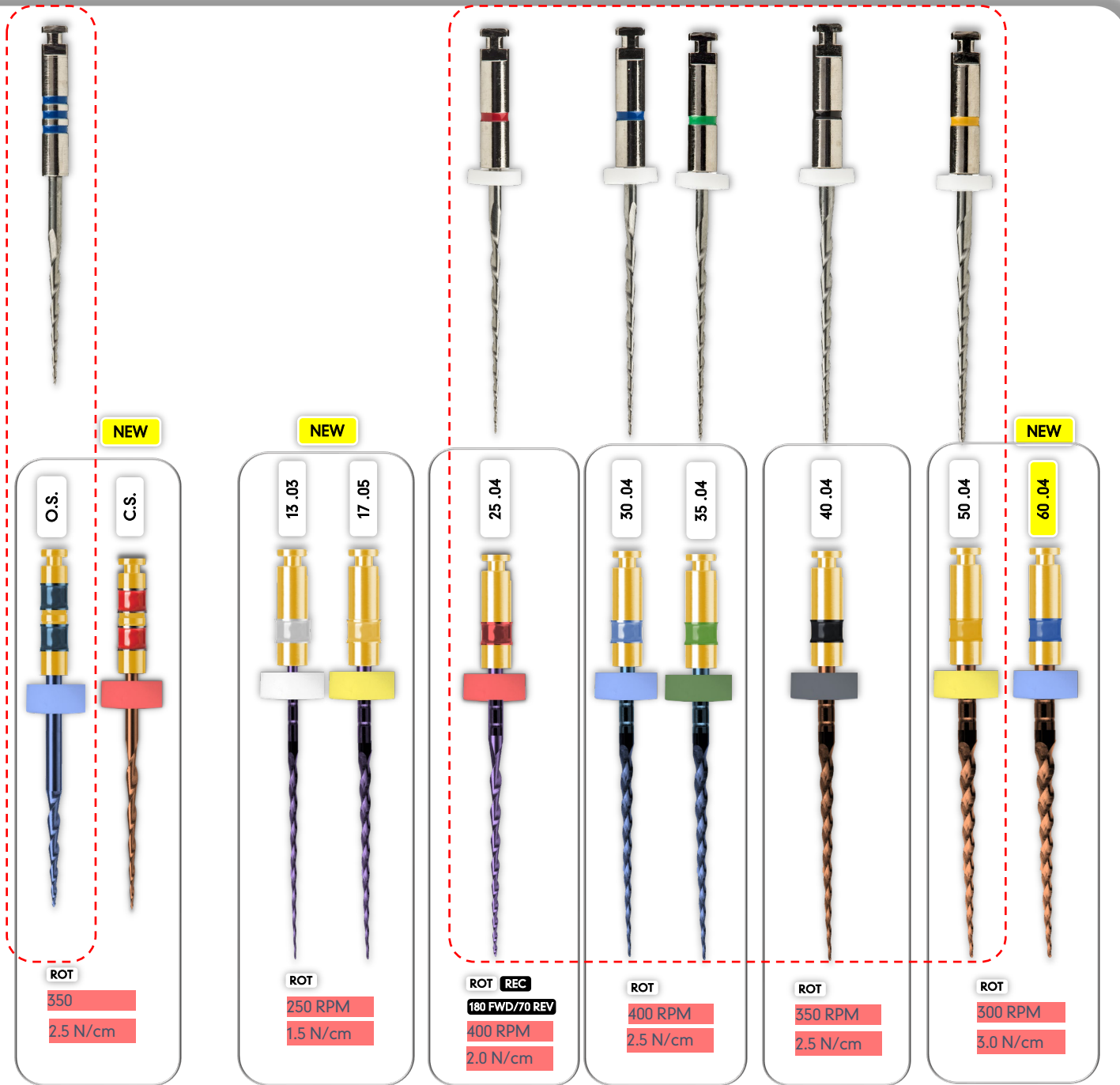

How to set up the Genius motor for Genius Proflex files?

1. Turn on the motor;
2. In ROTATING P1, press RPM, Plus or Minus to set 350;
3. Press Ncm button, Plus and Minus to set 1.5 torque;
3. Press the MODE button to RECIPROCATING GEN;
4. Press FWD button, Plus or Minus to set **180°**;
5. Press REV button, Plus or Minus to set **70°**;
6. Press RPM, Plus or Minus to set 400.

All files will work safely in the aforementioned parameters. If you want to be more specific on the rotations and torque for individualizing the file sequence **in rotation movement**, repeat 1-3 steps on ROTATING P2 as follows:

- For 25 .04: set RPM 400 and Torque 2.0;
- For 30/35 .04: set RPM 400 and Torque 2.0;
- For 40 .04 .04: set RPM 350, Torque 2.5;
- For 50/60 .04 .04: set RPM 350, Torque 2.5.

M Medidenta
Clinical Affairs

EDUCATION VIDEOS @

<https://www.medidenta.com/guides-maintenance/?cat=2>